

KN95 Mask with FDA Certificate

Product Name: KN95 Protective Mask

Package Size: 10 pcs/pkg



Shelf life: 3 Years

Standard: GB2626-2006


Carton Size: 520*380*560mm

80 packages per Carton



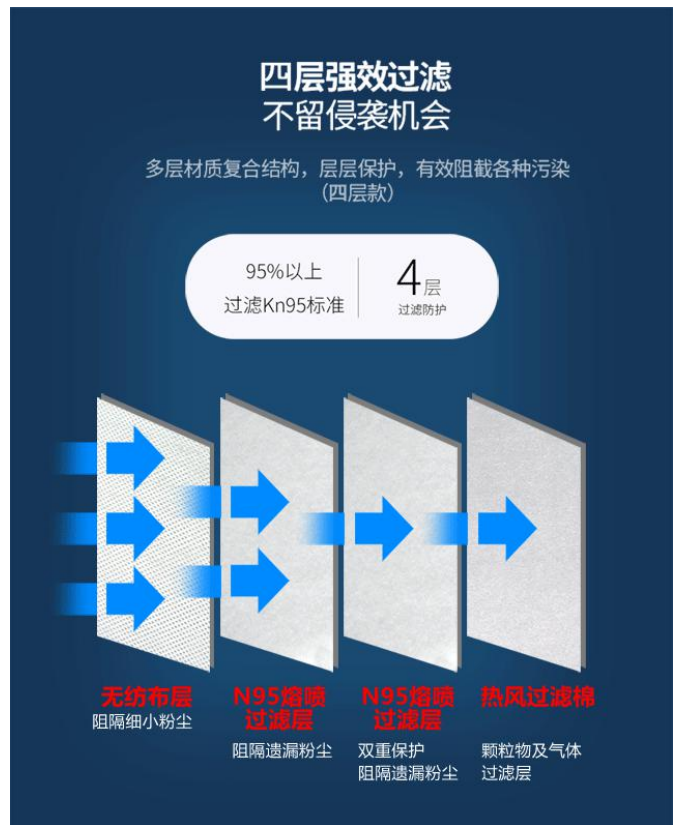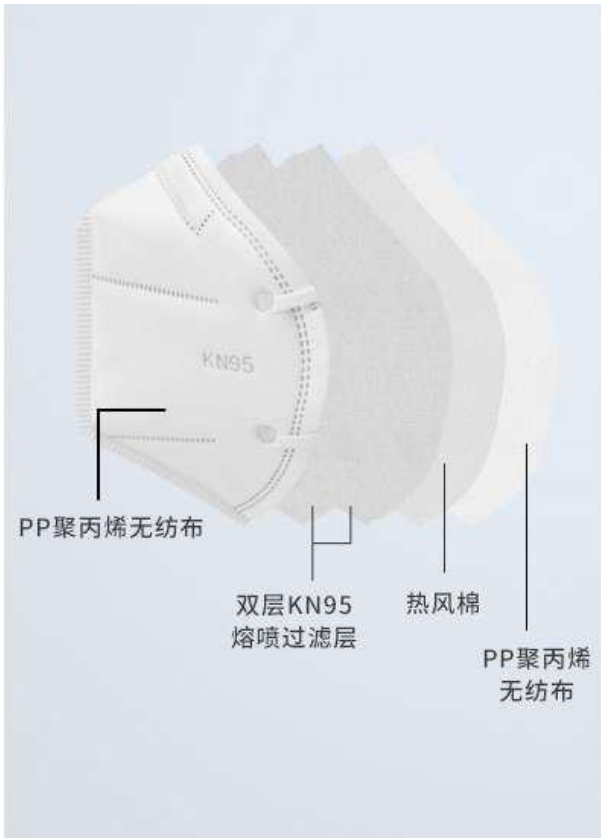

4 Layers, Non woven-two layers of N95 meltblown cloth-Non woven

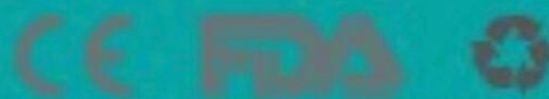

【品名】KN95防护口罩

[Name] KN95 protective masks

【产品尺寸】11.5*12cm

[Product size] 11.5 x 12cm

【包装规格】10只/盒

[Packaging specifications] 10 pcs / box

【生产批号 生产日期 失效日期】 见包装喷码处

[Production batch number Production date Expiry date]

【执行标准】GB2626-2006

[Executive standard] GB2626-2006

【贮存条件】温度<20度,无腐蚀性气体与通风良好的洁净室内。

[Storage conditions] temperature <20 degrees, clean room with no corrosive gas and good ventilation.

【产品结构及组成】由口罩体、鼻梁、口罩带组成。其中口罩体由内外层纺粘无纺布、中间层熔喷无纺布构成,共4层。产品为非无菌型,四层过滤结构,有效隔离异味与颗粒

[Product structure and composition] It consists of a mask body, a nose clip and a mask band. The mask body is composed of an inner and outer layer of spunbond non-woven fabric and a middle layer of melt-blown non-woven fabric, with a total of 4 layers. The product is non-sterile. Four-layer filter structure effectively isolates odor and particles

【有效防护时间】36小时

[Effective protection time] 36 hours

【生产企业名称】厦门福安堂药业有限公司

[Manufacturer] Fuman Tang Pharmaceutical Co., Ltd.

【住所】

【C

【Contact】

【邮政编码】361100

Postcode:361100

【售后服务单位】厦门福安堂药业有限公司

[After-sales service company] Xiamen

Fuman Tang Pharmaceutical Co., Ltd.



6 928388 919643



【使用方法】

- 1) 打开包装, 取出口罩, 握住口罩两头, 鼻梁条朝上;
- 2) 口罩遮住口鼻下颌, 耳带套在耳后;
- 3) 按压鼻梁条, 将其与鼻梁黏合;
- 4) 适当调整口罩位置, 使得佩戴舒适;
- 5) 按正反面正确佩戴, 手部请勿接触口罩内侧。

[Instructions]

- 1) Open the package, take out the mask, hold the two ends of the mask with the bridge of the nose facing upwards;
- 2) The mask covers the mouth, nose, and jaw, and the earband is placed behind the ear;
- 3) Press the bridge strip to fit the bridge of the nose;
- 4) Adjust the position of the mask appropriately to make it comfortable to wear;
- 5) Wear correctly according to the front and back sides. Do not touch the inside of the mask with your hands.

【注意事项、警示以及提示性内容】

- 1) 日常防护口罩有各种各样, 选择时必须针对不同的作业需求和工作条件。
- 2) 首先应根据粉尘的浓度和毒性选择, 根据GB/T18664《呼吸防护用品的选择、使用与维护》, 作为半面罩, 所有防尘口罩都适合有害物浓度不超过10倍的职业接触限值的环境, 否则就应使用全面罩或防护等级更高的呼吸器。
- 3) 如果颗粒物属于高毒性物质、致癌物和有放射性, 应选择过滤效率最高等级过滤材料。
- 4) 如果颗粒物具有油性, 务必选择适用的过滤材料。
- 5) 如果颗粒物为针状纤维, 如矿渣棉、石棉、玻璃纤维等, 由于防尘口罩不能水洗, 粘上微小纤维的口罩在面部密封部位易造成脸部刺激, 也不适合使用。

[Precautions, warnings, and informative content]

- 1) There are a variety of daily protective masks, which must be selected according to different operating requirements and working conditions.
- 2) Firstly, it should be selected according to the concentration and toxicity of dust, and according to GB / T18664 "Selection, Use and Maintenance of Respiratory Protective Equipment". As masks, all dust respirators are suitable for occupational exposure limits with a concentration of no more than 10 times the harmful substance. Environment, otherwise you should use a full face mask or a respirator with a higher degree of protection.
- 3) If the particulate matter belongs to highly toxic substances, carcinogens and radioactive materials, filter materials with the highest filtration efficiency should be selected.
- 4) If the particulate matter is oily, be sure to choose a suitable filter material.
- 5) If the particulate matter is needle-shaped fibers, such as slag wool, asbestos, glass fiber, etc., since the dust mask cannot be washed, a mask with microfibers on it can easily cause facial irritation at the face seal, and is not suitable for use.

CERTIFICATE OF FDA

Facility Registration and Device Listing of Medical Device

Pharmaceutical Co., Ltd

Owner/Operator Number

Registration/FEI Number: Active. Waiting for Registration Number Assignment

Address Industrial Park, Tong'an, Xiamen, NA

has completed the Facility Registration and Device Listing with the US FDA, through
HUMISS INC.

Listing Number	Premarket Submission	Product Code(s)	Device Name(s)	Activities	Models
	Exempt	MSH	Respirator, surgical	Manufacturer	Eagledon Disposable Medical Masks Eagledon Disposable Protective Masks Eagledon Disposable Surgical Mask, KN95, N95

HUMISS INC. will confirm that such registration remains effective upon request and presentation of this certificate until the end of calendar year stated above, unless said registration is terminated after issuance of this certificate. HUMISS INC. makes no other representations or warranties, nor does this certificate make any representation or warranties to any person or entity other than the named certificate holder, for whose sole benefit it is issued. This certificate does not denote endorsement or approval of the certificate-holder's product(s) or establishment by the U.S Food and Drug Administration. HUMISS INC. assumes no liability to any person or entity in connection with foregoing.

The U.S Food and Drug Administration does not issue a certificate of registrations, nor does the U.S Food and Drug Administration recognize a certificate of registration, HUMISS INC. is not affiliated with the U.S Food and Drug Administration.



Issue date: Mar. 23, 2020

Expire date: Dec. 31, 2020

Certi

HUMISS INC.
4845 Pearl East Cir Ste 118, Boulder, CO, 80301, USA
001-720-759-5888